


Sticky Whiskers™ Instructions

1. Prepare skin — clean and dry, free of any ointments, dressings
2. Wrap soft loop around cannula candy cane style as illustrated.

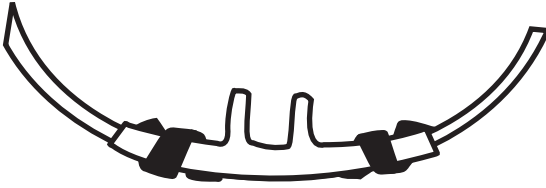

3. Place sticky whiskers along upper lip of infant, the ends can face upward or downward depending on the desired cannula placement.
Smooth in place.

4. Place prongs of cannula in infants nose and let the soft loop strips interface with the low profile hook on the sticky whiskers.



The sticky whiskers helps keep the cannula in place directly under the infant's nose. To change placement simply disengage the hook and loop and realign as desired. Sticky whiskers should be replaced if it becomes soiled or after 48-72 hrs. The hydrocolloid base is a gentle but secure attachment recommended by NANN in the skin care guidelines and will not cause skin irritation while in place or upon removal.

Caution: SINGLE PATIENT USE ONLY

DO NOT RESTERILIZE

DO NOT EXPOSE TO TEMPERATURES ABOVE 150 DEGREES F (65 C)

Contents of this package are clean and ready for normal use. They are not sterile.

Federal (USA) law restricts this device to sale by or on order of a physician.

For infant or pediatric use only

Country of origin: USA



Manufactured by

Beevers Mfg & Supply, Inc.
14670 NW Baker Creek Rd
McMinnville, OR 97128
United States of America
Tel: +1 (503) 472-9055
<http://www.beevers.net>



Authorized Representative


Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299



Explanation of symbols

	English	Manufacturer
	Français	Fabricant
	Deutsch	Hersteller
	Español	Fabricante
	Svenska	Fabrikör

	English	Authorized Representative
	Français	Représentant Autorisé
	Deutsch	Autorisierter Repräsentant
	Español	Representante Autorizado
	Svenska	Behörigt Representativt

	English	Consult Instructions for use
	Français	Consultez les instructions pour l'usage
	Deutsch	Beraten Sie Anwendungsvorschriften
	Español	Consulte las instrucciones para el uso
	Svenska	Rådfråga Instruktionerna för Använda